



ANDOVER HEALTHCARE



Wound Care Products

TABLE OF CONTENTS

| | |
|---------------------------------|-------|
| COMPANY PROFILE..... | 4 |
| VISUAL INDICATORS..... | 5 |
| PRODUCT SELECTION GUIDE..... | 6 |
| COMPRESSION GUIDELINES..... | 7 |
| COFLEX TLC DRY..... | 8-9 |
| COFLEX TLC LONG STRETCH..... | 10-11 |
| COFLEX TLC ZINC & CALAMINE..... | 12-13 |
| COFLEX AFD..... | 14 |
| COFLEX UMAFD..... | 15 |



Wound Care Products

FROM ANDOVER

Company Profile

Andover Healthcare is a leading manufacturer of high quality cohesive bandages and tapes. Founded in 1976, Andover is committed to new and innovative technology and distributes its patented products around the world.

All Andover products are made in the USA. Andover strives to develop break-through products to meet the ever-changing needs of the healthcare industry.

Andover has many industry “firsts” including developing:

- **A patented non-latex bandage**
- **A non-latex bandage that sticks over wet dressings**
- **Cohesive bandages with controlled compression**
- **Colors and color pack bandages**
- **Printed bandages**
- **Patented Hand-Tear cohesive bandages**

Product development is Andover’s specialty and customer service our commitment. At Andover Healthcare it’s not just about products, technology or service – it’s about all of these things, all of the time.



Introducing...




Visual Indicators

Now on ALL
CoFlex TLC Products

- ✓ Easy Application
- ✓ Intended continuous levels of compression



*Take the guesswork
out of application and
hit the target every time!*

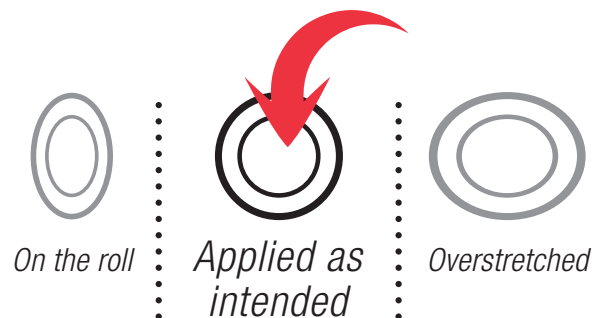


- Use visual indicators as a guide
- Stretch the bandage until the oval indicators become circles
- Visual indicators will ensure proper levels of compression based on the patient's ABI's

2 Layer Compression

PRODUCT SELECTION GUIDE

Andover Healthcare offers the widest range of two layer compression options for patients with oedema associated with venous insufficiency. The systems provide a two-step level of comfort and healing unlike any other Two Layer Compression or Unna Boot on the market. Choose from Lite or Standard Compression, in Dry or Moist (Zinc or Calamine). And now both Short Stretch and Long Stretch options with indicators!

- Lite = 25-30 mmHg / Standard = 35-40 mmHg
- Dry protects the skin / Moist (Zinc & Calamine) soothes the skin to promote healing
- Short Stretch for active patients / Long Stretch for less active patients

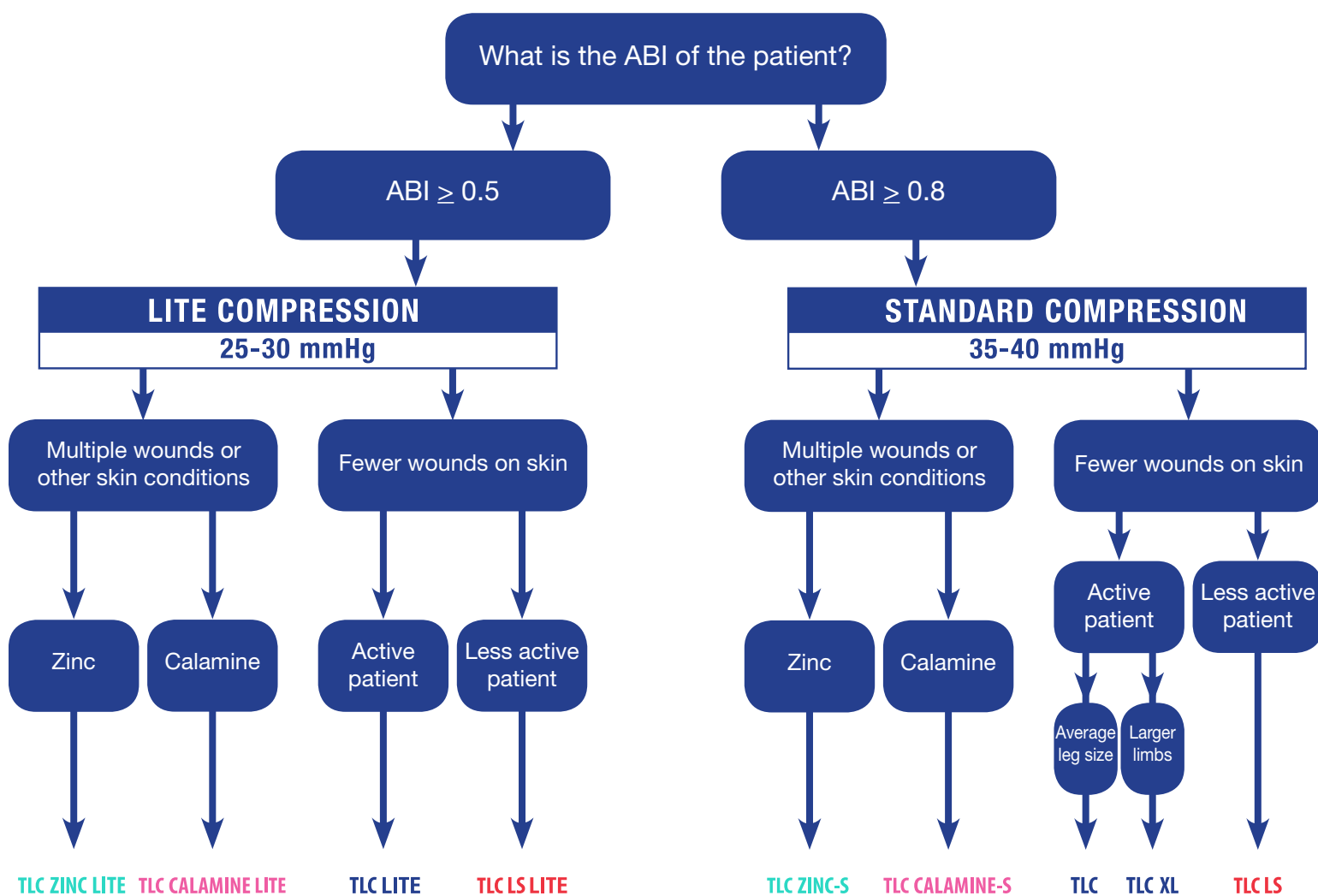

| | | LITE COMPRESSION 25-30 mmHg | STANDARD COMPRESSION 35-40 mmHg |
|-------------------------|-------|--|---|
| SHORT STRETCH | DRY | CoFlex·TLC LITE | DRY CoFlex·TLC CoFlex·TLC XL |
| | MOIST | CoFlex·TLC ZINC LITE CoFlex·TLC CALAMINE LITE | MOIST CoFlex·TLC ZINC-S CoFlex·TLC CALAMINE-S |
| LONG STRETCH | DRY | CoFlex·TLC LS LITE | DRY CoFlex·TLC LS |



CoFlex·TLC

2 LAYER COMPRESSION GUIDELINES

CoFlex® TLC

TWO LAYER COMPRESSION

DRY

NO LATEX

NEW
with
Indicators!

Two Layer Short Stretch

CoFlex® TLC delivers a two-step short stretch performance with high working pressure and low resting pressure. Visual indicators are provided for ease of application - ovals become circles when the intended compression is achieved.

Layer 1: Soft foam with odour and itch control properties. Excellent moisture transfer rate that wicks away drainage to help keep the peri-wound skin dry.

Layer 2: Non-latex short stretch compression bandage. Sticks to itself, Easy HandTear® Technology eliminating the need for scissors.



U.S. Patent No. 5,762,523, 6,156,424
European Patent No. 0839222
European & Canadian Patent Pending

Together the layers form a mechanical bond preventing slippage.



Use on active patients with oedema.

KEY FEATURES:

- Visual Indicators
- Itch Control
- Odour Control
- Easy Hand-Tear
- Nylon Stocking Included

Advantages:

- ✓ Easy to apply
- ✓ Comfortable & breathable
- ✓ Lightweight / low profile
- ✓ Sustained compression
- ✓ Non-slip support

Nylon Stocking Included

Apply nylon stocking over the completed dressing for patient comfort and ease-of-movement under clothes and on bed sheets



CoFlex®TLC

with Indicators

Each Box Contains 2 Rolls:
One of Each Layer + Nylon Stocking

STANDARD COMPRESSION
35-40 mmHg

For patients with an ABI of ≥ 0.8



| Catalog # | Color | Rolls/Box | Boxes/Case | Roll Name | Description | Roll Size |
|------------|-------|-----------|------------|-----------|-----------------------|-----------------------------|
| 7800TLC-TN | ● Tan | 2 | 8 | Layer 1 | Absorptive Foam Layer | 10 cm x 3,1 m (Unstretched) |
| | | | | Layer 2 | Compression Layer | 10 cm x 4,7 m (Stretched) |

CoFlex®TLC XL

with Indicators

Each Box Contains 2 Rolls:
One of Each Layer + Nylon Stocking

STANDARD COMPRESSION
35-40 mmHg

For patients with an ABI of ≥ 0.8



| Catalog # | Color | Rolls/Box | Boxes/Case | Roll Name | Description | Roll Size |
|------------|-------|-----------|------------|-----------|-----------------------|-----------------------------|
| 7800TLC-XL | ● Tan | 2 | 8 | Layer 1 | Absorptive Foam Layer | 10 cm x 4,9 m (Unstretched) |
| | | | | Layer 2 | Compression Layer | 10 cm x 6,4 m (Stretched) |

TLC XL has the same properties as standard CoFlex® TLC, but has an additional 2 yards on each roll for larger limbs.

CoFlex®TLC LITE

with Indicators

Each Box Contains 2 Rolls:
One of Each Layer + Nylon Stocking

LITE COMPRESSION
25-30 mmHg

For patients with an ABI of ≥ 0.5



| Catalog # | Color | Rolls/Box | Boxes/Case | Roll Name | Description | Roll Size |
|------------|-------|-----------|------------|-----------|-----------------------|-----------------------------|
| 7802TLC-TN | ● Tan | 2 | 8 | Layer 1 | Absorptive Foam Layer | 10 cm x 3,1 m (Unstretched) |
| | | | | Layer 2 | Compression Layer | 10 cm x 4,7 m (Stretched) |

TLC Lite has the same first layer as standard CoFlex® TLC with a lighter weight second layer compression bandage. The lower level of compression is designed for patients with an ABI ≥ 0.5 or those who are less tolerant of compression therapy.

CoFlex® TLC LS

NO LATEX

TWO LAYER LONG STRETCH COMPRESSION

DRY

NEW
with
Indicators!

Two Layer Long Stretch

CoFlex® TLC LS delivers a two-step long stretch performance with low working pressure and high resting pressure. Visual indicators are provided for ease of application - ovals become circles when the intended compression is achieved. Available in Standard and Lite.

Layer 1: Soft foam with odour and itch control properties. Excellent moisture transfer rate that wicks away drainage to help keep the peri-wound skin dry.

Layer 2: Non-latex long stretch compression bandage. Sticks to itself, Easy HandTear® Technology eliminating the need for scissors.

Together the layers form a mechanical bond preventing slippage.



Use on less active patients with oedema or where you have been using three or four layer kits.



U.S. Patent No. 5.762.523, 6.156.424
European Patent No. 0839222
European & Canadian Patent Pending

KEY FEATURES:

- Visual Indicators
- Itch Control
- Odour Control
- Nylon Stocking Included
- Easy Hand-Tear

Advantages:

- ✓ Easy to apply
- ✓ Comfortable & breathable
- ✓ Lightweight / low profile
- ✓ Restorative compression
- ✓ Non-slip support

Nylon Stocking Included

Apply nylon stocking over the completed dressing for patient comfort and ease-of-movement under clothes and on bed sheets



Available in Lite and Standard compressions.



CoFlex®-TLC LS

with Indicators

Each Box Contains 2 Rolls:
One of Each Layer + Nylon Stocking

STANDARD COMPRESSION
35-40 mmHg

For patients with an ABI of ≥ 0.8

| Catalog # | Color | Rolls/Box | Boxes/Case | Roll Name | Description | Roll Size |
|------------|-------|-----------|------------|-----------|-----------------------|-----------------------------|
| 7900TLC-TN | ● Tan | 2 | 8 | Layer 1 | Absorptive Foam Layer | 10 cm x 4,3 m (Unstretched) |
| | | | | Layer 2 | Compression Layer | 10 cm x 7,3 m (Stretched) |



CoFlex®-TLC LS LITE

with Indicators

Each Box Contains 2 Rolls:
One of Each Layer + Nylon Stocking

LITE COMPRESSION
25-30 mmHg

For patients with an ABI of ≥ 0.5

| Catalog # | Color | Rolls/Box | Boxes/Case | Roll Name | Description | Roll Size |
|------------|-------|-----------|------------|-----------|-----------------------|-----------------------------|
| 7902TLC-TN | ● Tan | 2 | 8 | Layer 1 | Absorptive Foam Layer | 10 cm x 4,3 m (Unstretched) |
| | | | | Layer 2 | Compression Layer | 10 cm x 7,3 m (Stretched) |

TLC LS Lite has the same first layer as standard CoFlex® TLC LS with a lighter weight second layer compression bandage. The lower level of compression is designed for patients with an ABI ≥ 0.5 or those who are less tolerant of compression therapy.

TWO STEP EASY APPLICATION



CoFlex® TLC

TWO LAYER COMPRESSION

ZINC & CALAMINE

NO
LATEX

NEW
with
Indicators!

Two Layer Short Stretch with Zinc or Calamine

CoFlex® TLC Zinc & Calamine delivers a two-step short stretch performance with calming benefits of zinc or calamine. This unique system combines the benefits of a traditional Unna Boot with compression therapy in a much cleaner two-step process. Visual indicators are provided for ease of application - ovals become circles when the intended compression is achieved.

Layer 1: Soft foam impregnated with zinc or calamine. Soothes and calms skin with multiple wounds or other skin conditions.

Layer 2: Non-latex short stretch compression bandage. Sticks to itself, Easy HandTear® Technology eliminating the need for scissors.

Together the layers form a mechanical bond preventing slippage.



U.S. Patent No. 5,762,523, 6,156,424

European Patent No. 0839222

European & Canadian Patent Pending

KEY FEATURES:

- Visual Indicators
- Soothing Zinc or Calamine
- No Unna's Boot Mess
- Nylon Stocking Included
- Easy Hand-Tear
- Moisture Resistant Controlled Compression

 Use on patients with oedema and multiple wounds or skin conditions.

Advantages:

- ✓ Easy to apply
- ✓ Comfortable & breathable
- ✓ Absorbs 20x's its dry weight vs. traditional Unna Boot
- ✓ 50% more active ingredients than traditional Unna Boot
- ✓ Non-slip support

Nylon Stocking Included

Apply nylon stocking over the completed dressing for patient comfort and ease-of-movement under clothes and on bed sheets



CoFlex[®]·TLC ZINC

with Indicators



| | Catalog # | Color | Boxes/Case | Roll Name | Description | Roll Size |
|--|------------|-------|------------|-----------|-------------------|--------------------------------------|
| LITE Compression <i>For patients with an ABI of ≥ 0.5</i> | 8840UBZ-TN | ● Tan | 8 | Layer 1 | Zinc Foam Layer | 10 cm x 5,5 m (<i>Unstretched</i>) |
| | | | | Layer 2 | Compression Layer | 10 cm x 6,4 m (<i>Stretched</i>) |
| Available in 3" = 8830UBZ-TN | | | | | | |
| STANDARD Compression <i>For patients with an ABI of ≥ 0.8</i> | 8840UBZ-SC | ● Tan | 8 | Layer 1 | Zinc Foam Layer | 10 cm x 5,5 m (<i>Unstretched</i>) |
| | | | | Layer 2 | Compression Layer | 10 cm x 6,4 m (<i>Stretched</i>) |
| Available in 3" = 8830UBZ-SC | | | | | | |

CoFlex[®]·TLC CALAMINE

with Indicators



| | Catalog # | Color | Boxes/Case | Roll Name | Description | Roll Size |
|--|------------|-------|------------|-----------|---------------------|--------------------------------------|
| LITE Compression <i>For patients with an ABI of ≥ 0.5</i> | 8840UBC-TN | ● Tan | 8 | Layer 1 | Calamine Foam Layer | 10 cm x 5,5 m (<i>Unstretched</i>) |
| | | | | Layer 2 | Compression Layer | 10 cm x 6,4 m (<i>Stretched</i>) |
| Available in 3" = 8830UBC-TN | | | | | | |
| STANDARD Compression <i>For patients with an ABI of ≥ 0.8</i> | 8840UBC-SC | ● Tan | 8 | Layer 1 | Calamine Foam Layer | 10 cm x 5,5 m (<i>Unstretched</i>) |
| | | | | Layer 2 | Compression Layer | 10 cm x 6,4 m (<i>Stretched</i>) |
| Available in 3" = 8830UBC-SC | | | | | | |

“

Finally, a new Unna boot!

Dr. Larry Chase, MD, CWS, FCCWS
CoxHealth Hyperbaric Medicine and Wound Care Center
Springfield, MO

“

My patients are pleased with the TLC Zinc and Calamine, especially when they see how much edema is reduced. They're easy to apply and provide good compression - they give immediate comfort.

Dr. Joseph Snow, MD, Wound Care Specialist
Concord Hospital, Wound Healing Center
Concord, NH

“

Our patients and staff love the CoFlex Unna Boot! Patients tell us CoFlex is comfortable. They don't slide or feel slimy like other products they have used. Our nurses love it for the ease and efficiency in application.

Julie Darmon, Nurse Manager
Barnes Jewish Hospital WCC
St. Louis, MO

Each Box Contains 2 Rolls - One of Each Layer + Nylon Stocking

CoFlex® AFD

ABSORBENT FOAM DRESSING



All-in-One Foam Pad Dressing & Bandage

CoFlex® AFD's highly absorbent foam pad is attached to soft, quick-stick CoFlex® LF² for easy application. Between the LF² layer and the absorbent foam is a waterproof film layer that prevents leakage, drying out and contamination. Also available sterile.



Key Features:

- ✓ Quick & easy to apply
- ✓ High absorption and retention of fluids (equivalent to 29 gauze pads)
- ✓ Non-adherent foam pad conforms but won't stick to wound bed
- ✓ Flexible, controlled compression
- ✓ Made in the USA

PRODUCT APPLICATIONS:

- Trauma Bandaging
- Post-op cranial surgery
- General wound dressing
- Cover/protect post surgery incisions
- Post bunion surgery

| Catalog # | Size | Rolls/Case | Catalog # | Size | Rolls/Case | Colors |
|------------|----------------|------------|------------|------------------|------------|--------|
| 9720TN-016 | 5 cm x 2,28 m | 16 | 9760TN-008 | 15,2 cm x 2,28 m | 8 | ● |
| 9720(S) | | | 9760(S) | | | |
| 9740TN-008 | 10 cm x 2,28 m | 8 | 9840TN-006 | 10 cm x 6,85 m | 6 | ● |
| 9740(S) | | | | | | |

Choose From: Tan (TN), Sterile Tan (S)

When ordering, please indicate catalog # and color code
 For example, 9720TN-016 (XX = color code)
 Most products are available in bulk packaging, call for details.

CoFlex® + UMAFD



UNIVERSAL MEDIC ABSORBENT FOAM DRESSING

All-in-One Trauma/Combat Bandage

CoFlex® UMAFD is a strong bandage with foam pad attached, plus an additional foam pad in the core for extra absorbency. It offers quick, easy application in high stress situations. It is a great alternative to existing combat bandages. Additionally, CoFlex® UMAFD has been heat tested at 130° > 90 days.

Key Features:

- ✓ Quick & easy to apply
- ✓ High absorption and retention of fluids (equivalent to 29 gauze pads)
- ✓ Non-adherent foam pad conforms but won't stick to wound bed
- ✓ Flexible, controlled compression
- ✓ Able to withstand high temperatures
- ✓ Made in the USA



| Catalog # | Size | Rolls/Case | Colors |
|------------|----------------|------------|--------|
| 9340UM-018 | 10 cm x 2,74 m | 18 | ● |
| 9340UM-100 | 10 cm x 2,74 m | 100 | ● |

Available in Black only

Durable trauma bandage for situations requiring additional wound packing, strength for compression, and high heat/water resistance for any of the following:

- Military - Medics, Soldiers
- ER / Trauma Centers
- First Responders
- Commercial & Industrial Sites
- Emergency Medical Services
- First-Aid Kits



We were very happy with how easy the CoFlex AFD foam pad dressing was to apply. Patients we sent home with the product were able to change the dressings without difficulty and therefore more compliant. We used the CoFlex AFD foam pad dressing on many types of wounds and were pleased with the healing we saw with them all.

Amy Mahoney, Wound Care Specialist
Boone Hospital, Wound Clinic
Columbia, MO



ANDOVER HEALTHCARE, INC.
9 FANARAS DRIVE, SALISBURY, MA 01952
PHONE: 978.465.0044 TOLL FREE: 800.432.6686
FAX: 978.462.0003

www.andoverhealthcare.com